9 3 16 5 6 12 4 13 11

Inhaltsverzeichnis - [Contents](#)

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
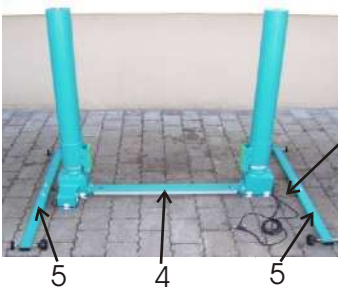
1. Insert and screw the cross beam (4) to the lifting column drive housings (16) (identical)

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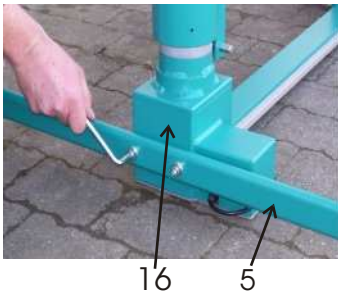2. Lay the cables from the drives to the control box side (to the right as seen from the bottom front)

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
3. Screw the side cantilevers (5) to the drive housings (16)

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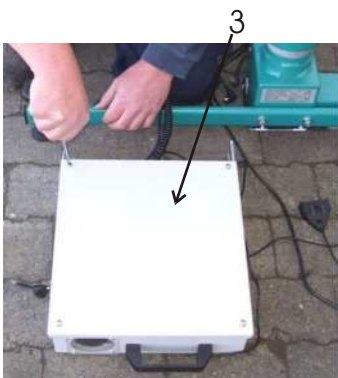4. Vertically level the lifting columns (1) with the help of the screws (6), assuring that the drive housings always remains in contact with the ground

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5. Set the control box (3) onto the right-hand cantilever (5) frontpart or backpart. Unscrew the cover and plug the two drive cables into the processor control box-M1, M2 (allocation as desired). Inspect preassembled connections and close the cover

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General

On-site assembly and setup must be carried out in accordance with recognised technical procedures, and it is assumed that installation personnel have been appropriately trained and qualified!

Required tools (not included in scope of delivery):

- 10 and 13 mm combination spanners
- 8 and 10 mm hex wrenches
- Size 2 Phillips screwdriver

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Only the two drive cables need to be disconnected for dismantling

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6. Insert the platform: Set the platform down onto the holder blocks

Auffahrklappe (11) frei beweglich montieren (falls separat geliefert)
Mount the drive-on ramp (11) such that it can be freely moved (if shipped loose)

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7. Secure with handscrews (1 on each side)

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8. Plug the coil cable into the safety flap (13) (socket at left)

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9. Insert the folding stirrup (7) towards the access down

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Adjust to appropriate height and tighten with the hand screws

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10. Final inspection:

- Secure, skid-proof stance (in case of permanent installation the cross beam (4) must be bolted to the ground by means of the mounting holes furnished to this end)
- Correct laying and securing of the cables
- Free travel path without any points at which jamming or shearing can occur

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For operation by hand or for surface mounting, without stop position memory

The AUER PL-08 portable lift is equipped with ultramodern processor controls including so-called first-fault safety, and complies with the strictest standards which apply, for example, in hospitals and for nursing services. All functions are continuously monitored and a first fault, or malfunctioning, can never cause a hazard. This eliminates the need for fuses and an emergency stop switch, although an emergency stop switch can be connected if desired (option).

The maintenance-free buffer battery supplied as an optional feature allows for approximately 20 lifting operations at reduced speed without mains power.

The large safety flaps underneath the platform cause the system to stop immediately upon contact with any obstacle, and the platform is automatically reversed for a distance of approximately 5 mm.

The start command is triggered by means of the hand switch with cable using the UP and DOWN keys, or by the cable handset with memory button. The handset can be wall mounted at the bottom stop position. The handset has a memory function for 4 stop positions which can be overwritten as desired.

An external safety contact can also be connected, e.g. for a gate at the installation site.

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Stop positions 1-4 to save by memory button "S":

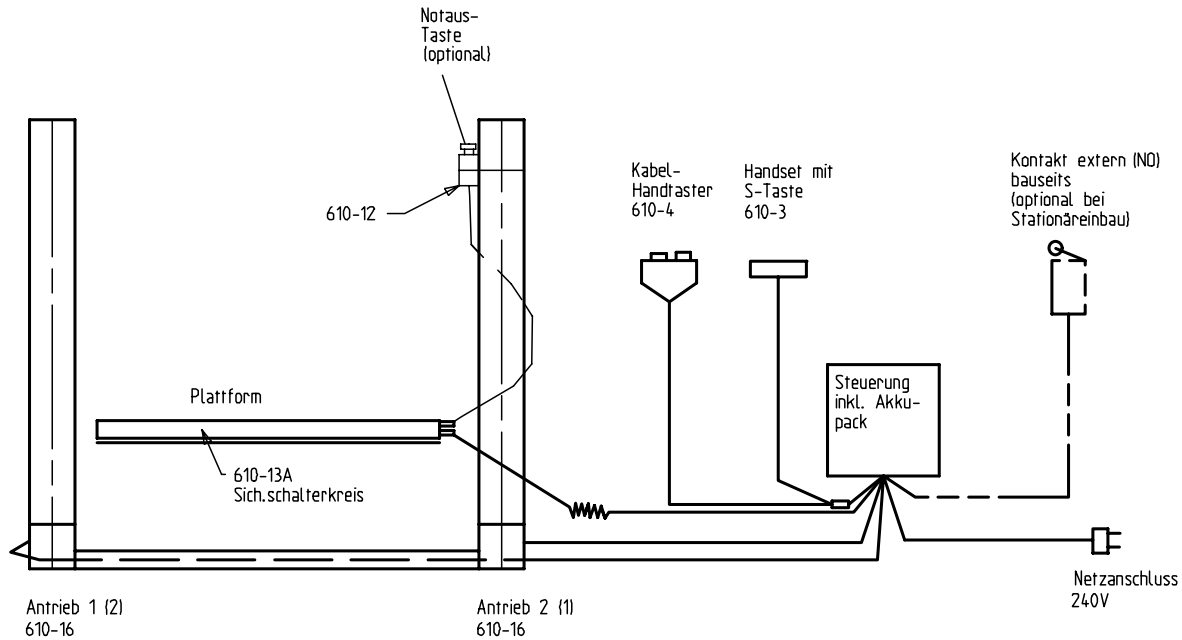
*1. Move platform to desired height
2. Press "S" button and within 5 sec.
3. desired button 1-4*

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Important Information



- *The AUER-PL-08 portable lift is designed for temporary or permanent installation for a sill height of up to 770 mm.*
- *Maximum load capacity is 300 kg of symmetrical load distribution.*
- *The PL-08 is intended for the transport of one standing or in a wheelchair seating person. It is not intended, nor is it suited for transporting goods or any other purposes.*
- *Maximum duration of use is limited to 10% of any given unit of time (e.g. 6 minutes during a period of 1 hour).*
- *Prevent unauthorised manipulation by children and other unauthorised persons, because of dangerous consequences might otherwise result!*
- *Keep this user's manual on hand at all times, and adhere to all operating instructions included therein for your own safety.*
- *The surface on which the lift is set up must be level, and must be furnished with a robust covering.*
- *The wall to be traversed must have a smooth, perpendicular surface without any protrusions which might result in the risk of jamming, pinching or shearing.*
- *Adequate clearance of approximately 20 mm must be assured between the lifting platform and the wall.*
- *Any possible slipping of the lift (e.g. due to vibration) must be prevented with appropriate means. If the lift is installed permanently, it must be bolted to the ground. A permanent outdoor use is allowed only with the optional equipment wetland protection FSA (otherwise no guarantee)!*
- *If unusual noises, motion, malfunctions or damage should become apparent, the system must be removed from service immediately. The system may not be placed back into service until it has been subjected to the detailed inspection and trouble-shooting by an authorised service provider!*
- *Locally applicable regulations must be adhered to.*
- *The manufacturer and the supplier of the lift assume no liability and accept no guarantee claims in the event of non-compliance with or disregard of the operating instructions included in this user's manual.*
- *The maximum travel limits are works calibrated. For a new calibration p.e. because of replacing of lifting columns or controls a special calibration set for PC can be delivered.*

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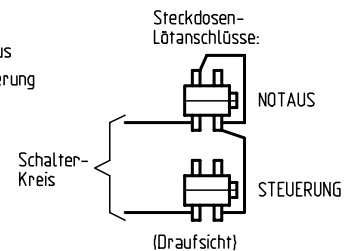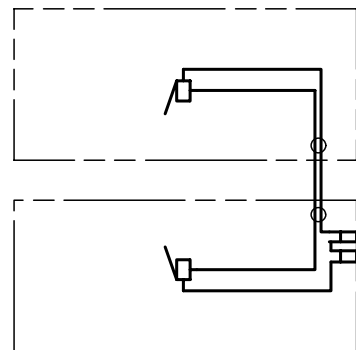Wartungskontrolle, Reparaturen

Maintenance

The drives and the controls included with the AUER PL-08 portable lift are maintenance-free for the entire service life of product.

Maintenance procedures for lift components:

1. Daily, i.e. after each use:
*Visual inspection of the system, in particular:
Position, safety devices, cable routing, obstacles in
the travel path, cleanliness etc.*

2. Semi-annual:
*Same as point 1. plus:
- Maintenance of lifting columns*

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*- Treatment of joints with oil spray
- Treatment of painted surfaces with silicon spray*

3. Annual (semi-annual in case of heavy-duty use):
*Same as points 1. and 2. plus:
Complete system inspection and maintenance
performed by an authorised service provider.*

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*** Remark:**

Reset run requires also to loose one of the platform fixations!

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Model: PL-08

Serial numbers: 0010 - 0100

Declaration: I declare that as the authorised representative, the above information in relation to the supply of this product, is in conformity with the stated standards and other related documents following the provisions of the above directives and their amendments.
Condition for the validity of this declaration in foreign countries is a translation of the manual into a local language.

Date:

Signature of the
authorised
representative:

Peter Auer, CEO